

TruTone Emote Instructions



1. Remove all items from the package and remove plastic coverings.
2. There should be 7 items total:

a. Body




b. Sound Heads (x2)



c. Oral Adapter



d. Oral Tubes (x4)

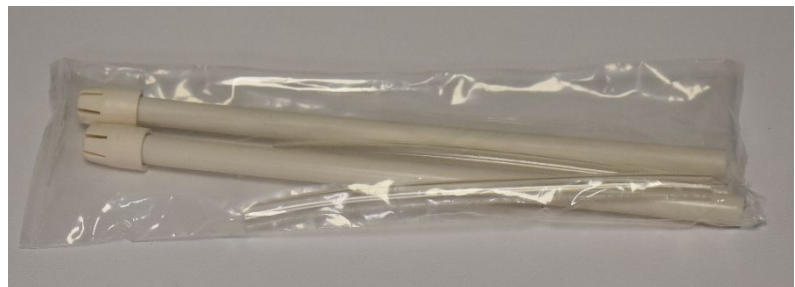

e. USB Charging Adapter



3. Insert the Battery at the bottom of the Body. Then, use the charger to charge once or twice a week. When the product is charging, a LED light will turn on, and once it finished, the light will flash.
4. Select your desired Sound Head and screw it on to the top of the Body. There are two kinds of Sound Heads:
 - a. The Sound Head with a black top is the normal attachment.
 - b. The Sound Head with a grey top is an Extra Sharp Sound-Head Cap. This does not sound as clear but can help in loud environments.

5. Next, adjust the Base-Tone. Turn the knob up for a higher tone, and down for a lower tone.

6. Set the MODE. The Mode button is located underneath the battery cap. To change the mode, you must press the Mode button and the Power button at the same time. There are 6 Modes:

1. Mode 1 is Monotone. There will be 1 beep to indicate this mode.

2. Mode 2 is low tonal variations. There will be 2 beeps.

3. Mode 3 is medium tonal variations. 3 beeps will play.

4. Mode 4 is high tonal variations. There will be 4 beeps.

i. Modes 5-6 are volume modes. These allow you to control the volume with the power button. A light touch will sound like a whisper and a firm touch will emit full volume.

5. Mode 5 is low sensitivity- full volume requires more pressure. There will be 2 “whoop” sounds.

6. Mode 6 is high sensitivity- full volume requires less pressure. There will be 3 “whoops”.

7. Neck placements tips:

- a. Speak as sound carries into your mouth. Do not force air out of your mouth.
- b. Make sure the Sound-Head is placed completely against your neck.
- c. Try different positions on your neck until the best sound is produced.

8. To use an Oral Adapter and Oral Tubes:

- a. The Oral Adapter goes on top of the Space-Head. Then, place an Oral Tubes inside the Oral Adapter. Place the tube in the corner of your mouth and speak around it.

How to Use:

A. Sound-Head

-This goes against the neck and vibrates to create sounds.

B. Sound-Head Spacer

-This can be removed for a clearer sound if a high tone is used frequently

C. Power Button

-This activates the base tone

D. Volume Knob

- This turns the volume up and down

E. Body

F. Battery

K. Tone Adjustment Knob

- This turns the tone higher and lower.

LED. LED Indicator Light

- This light turns on if there is low battery.

- It stays on when charging and flashes when charging is complete.

M. MODE Button

- There are 6 modes that change the tonal ranges.

